

INSTRUCTIONS FOR TRANSPORT BOXES

Ready for despatch in only 6 steps

1 Transport box

Please use the transport box for storage and transport of medical devices used on patients.
Enclosures/contents: Hand-over note, transparent plastic bags for packaging the devices plus seals for closing the transport box.

2 Preparation of medical devices

Please clean medical devices according to instructions for preparation in hospital.

3 Packaging and labelling

Pack each pre-cleaned medical device individually in the provided transparent plastic bag, together with respective *manufacturer's original data / original label. Check the transport box for completeness:

- Medical devices packaging Hand-over
- Note Manufacturer's original data /
- Original label *

*ATTENTION

Manufacturer's original data / original label applies only to initial submission of medical devices. No reprocessing is possible without the original manufacturer's label.

4 Hand-over note

Please ensure that a signed hand-over note is enclosed with the transport box.

5 Sealing the transport box

Close the lid of the transport box and push each supplied seal into openings provided on the left and right front of the closed box. The transport box is now ready for collection.

6 Shipping

Place the green shipping box into the white plastic bag and seal the latter. The VANGUARD shipping box is now ready for collection.